

# Available Now

# Oratect®

The first **FDA 510K  
CLEARED** On-site  
Saliva Test.

Made in USA

US Patent: 6248598

**FDA 510K CLEARED**

- Two test strip configurations of 6 drugs
- Rapid collection and testing in one step
- Results in 5 minutes
- User-friendly and non-invasive
- Avoids cross gender observation
- Reduces sample adulteration or contamination
- Includes transport tube for confirmation testing

### **Drugs of Abuse**

Methamphetamines (ME)	50 ng/mL
Marijuana (TH)	40 ng/mL
Cocaine (CO)	20 ng/mL
Amphetamines (AM)	50 ng/mL
Opiates (OP)	40 ng/mL
Phencyclidine (PC)	10 ng/mL





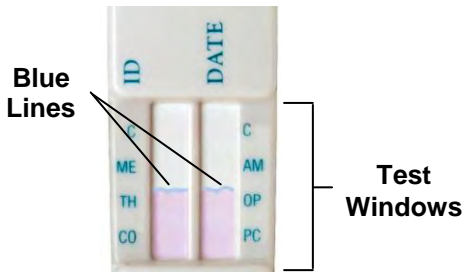





## Oratect® Reference Guide

The Oratect® Oral Fluid Drug Screen Device is a simple one-step test for the detection of drugs of abuse in oral fluid.

**Oratect® Procedures** -- Refer to package insert for detailed instructions and technical information.

<p>1. <b>Remove</b> the blue cap by holding the sides and pulling gently. This will expose the collection pad. <b>Make sure there is a blue line present in each window area indicating an unused device.</b></p> 	<p>2. <b>Open</b> mouth and gently <b>rub</b> the collection pad inside mouth against cheek in a circular motion several (approximately 15-20) times. Make sure to <b>keep head level.</b></p> 	<p>3. Gently <b>rub</b> the collection pad against the opposite cheek in circular motion several (approximately 15-20) times.</p> 
<p>4. Gently <b>rub</b> the collection pad <b>on top</b> of the tongue several (approximately 15-20) times. <b>Do not chew, suck, bite or bend the collection pad.</b></p> 	<p>6. Place the collection pad underneath the tongue for approximately 30 seconds to collect saliva. Instruct the donor to hold the device in place with hand. <b>Sufficient amount of saliva collected is indicated by the flow of the blue lines. Repeat steps 2-6 until blue lines flow.</b></p> 	<p>7. <b>Remove</b> from mouth as soon as blue lines flow at both of the test windows. <b>Re-cap</b> the device.</p> 
<p>5. <b>Rub</b> the collection pad <b>underneath</b> the tongue several (approximately 15-20) times.</p>		<p>8. Lay the device on a flat surface and <b>read results in 5 minutes after removing the device from mouth. See Package Insert.</b></p>

### Interpreting Oratect® Test Results

#### Invalid Result

When **no** colored band appears in the **CONTROL (C) region**, the test is **invalid** even if there is a band in the test region. **Repeat** the test with a new device.



Example Interpretation:

ME: Invalid  
TH: Invalid  
CO: Invalid  
AM: Invalid  
OP: Invalid  
PC: Invalid

#### Negative Result

For each test, two colored bands should be observed:

- **One in the CONTROL (C) region**
- **One in the specific TEST region**

The color of the test band may be slightly darker or lighter than the control band. Any visible band that can be seen is a **negative** result.



Example Interpretation:

ME: Negative  
TH: Negative  
CO: Negative  
AM: Negative  
OP: Negative  
PC: Negative

#### Presumptive Positive Result

A colored band at the **CONTROL (C) region** should be observed. When there is no colored band at the specific **TEST region**, the test is presumptive positive for that particular drug.



Example Interpretation:

ME: **Presumptive Positive**  
TH: Negative  
CO: Negative  
AM: Negative  
OP: Negative  
PC: Negative