

ProScreen Cassette Instructions



✓ **STEP 1**

Collect Specimen




✓ **STEP 2**

**Verify temperature is
within range (32-38 C)**



✓ **STEP 3**

**Apply 3 drops of specimen to
each sample well**

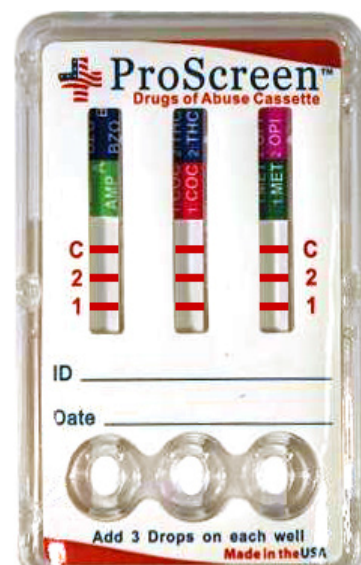

✓ **STEP 4**

**Test will run in 1-2 minutes.
Read result when lines appear.**

ProScreen Results Interpretation

Reading the Result Windows



NEGATIVE

A Negative test is indicated by the presence of a test line below each designated drug.



NEGATIVE

This is also a Negative screen. Even a **very light** colored line indicates a negative result.



POSITIVE

A Positive test is indicated by the **ABSENCE** of a Test Line below the designated drug, with the Control line present for that channel.



INVALID

An Invalid test is indicated when the "C" Control Lines are completely missing in one or more of the test windows. In event of an invalid screen a second screen should be run.

Wait 5 minutes before reading Positive Results. Do not read results Past 10 Minutes.