




### ✓STEP 1

Collect Specimen and verify temperature is within range (32-38 C)



### ✓STEP 2

Remove lower cap and dip test fingers into specimen

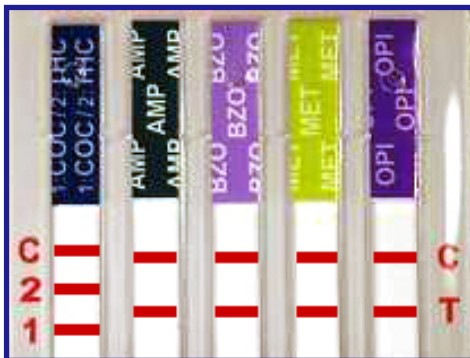

### ✓STEP 3

Once all windows begin to turn pink, remove ProScreen Dip from the specimen and replace the cap.




### ✓STEP 4

Interpret Adulteration Strip and Test Results


# ProScreen Results Interpretation

## Reading the Result Windows

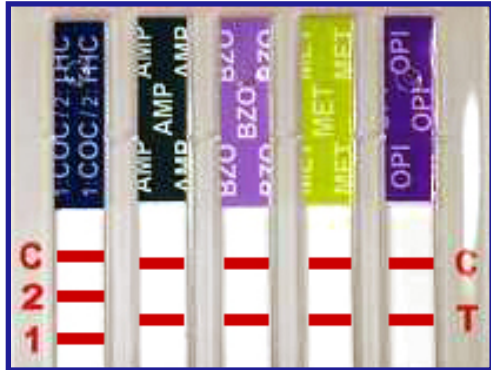


**NEGATIVE**

A Negative test is indicated by the presence of a test line below each designated drug.

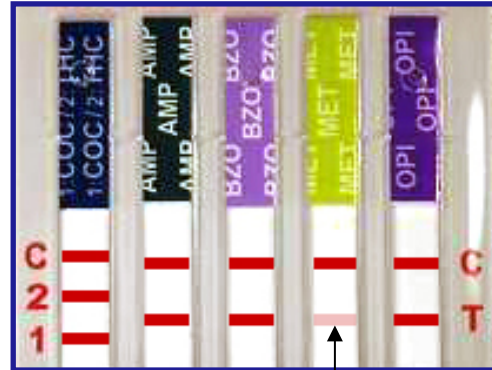


**NEGATIVE**

This is also a Negative screen. Even a **very light** colored line indicates a negative result.

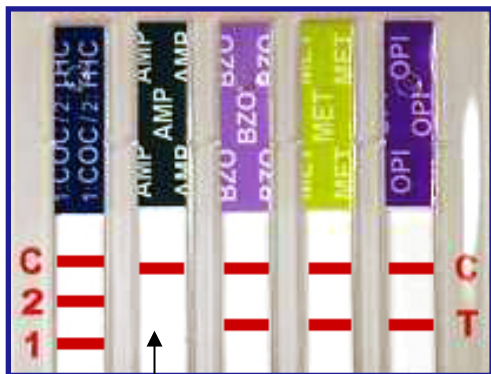

**POSITIVE**

A Positive test is indicated by the **ABSENCE** of a Test Line below the designated drug, with the Control line present for that channel.

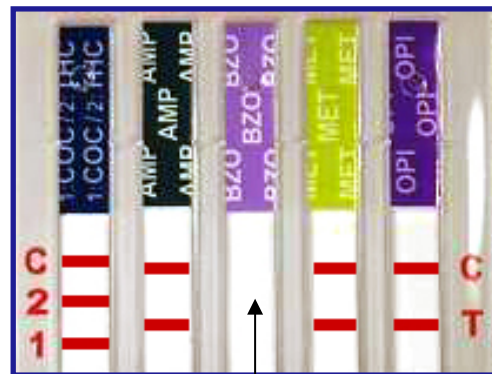

**INVALID**

An Invalid test is indicated when the "C" Control Lines are completely missing in one or more of the test windows. In event of an invalid screen a second screen should be run.

**Wait 5 minutes before reading Positive Results. Do not read results Past 10 Minutes.**